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8

Indications for use

Post-operative care, analgesic treatment, sports trauma (sprain, tear, etc...), after rehabilitation session...

Contraindications

Allergy to cold, insensibility to cold, Raynaud's syndrome, cryoglobulinemia, arteritis, diabetes, polyarthritis...

Precautions for use

The manufacturer makes no guarantee that the use of this product will prevent the occurrence of an injury. If user presents with characteristics that promote the risk of skin burns (e.g. malnourished patients, large smokers, fragile skin, insulin-dependent diabetics, etc.), close medical supervision is required and recommended.

This medical device should be used as prescribed by your surgeon or doctor or according to the recommendations for use of your duly authorized health professional (e.g. pharmacist, physiotherapist, orthotist).

Cold packs should never come into direct contact with the skin.

Do not puncture the cold packs. In the event of a leak, do not pour the gel into a sewer-connected drainage system or into a stream. For more information, contact Orthonov or visit www.orthonov.com

If the gel is in direct contact with the skin, wash immediately and thoroughly with water. Contact your doctor if you have persistent irritation.

Keep out of the reach of children or the elderly unable to use this device.

Instructions

1. Insert the cold packs in a dry, airtight freezer bag, before putting them in the freezer. After 8 hours of freezing (between -18 and -22 ° C), take out the cold packs and distribute the gel if necessary.

2. Place the posterior cold pack on a flat surface. The longest straps will be used for the thigh.

3. Put on the lycra cloth and place the leg in order to center the back of the knee on the posterior cold pack from a above point of view.

4. Place the anterior cold pack on the front of the knee. The green border must be on the thigh.

5. Fold down the posterior cold pack so that it overlaps the anterior cold pack and fix the velcro. If necessary, use the extensions.

6. From now on, the leg must remain straight, if possible ankle above the knee level.

7. Turn the tap to ON and screw on the pump. Inflate (do not exceed 15 pressures). Turn the tap to OFF and unscrew the pump.

8. After 20 min, turn the tap to the ON position to release the remaining air and undo the straps. Wide and replace the cold packs in the bag and store in the freezer.

If you have any questions about using this device, please contact Orthonov.

Maintenance

The cold packs are washable by hand with a mild soap.

Do not immerse them in a bath of disinfectant detergent but spray them with a disinfectant detergent solution (ANIOS high surface disinfectant detergent is recommended). Ensure the tap is in the OFF position.

WARNING

In case of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your doctor, surgeon, or authorized health professional (e.g. pharmacist, physiotherapist, orthotist) as soon as possible, and contact Orthonov: +33 1 60 13 65 73 or contact@orthonov.paris

Manufacturer

Caution,
Consult Accompanying
Documents

Hand Wash Only

Do Not
Tumble Dry

Do Not Iron

Do Not Bleach

Do Not Cut

CE Mark

3 rue de Villebon - 91160 Saulx-les-Chartreux - France
Tél.: +33 1 60 13 65 73 - Fax: +33 1 69 15 06 37
Mail: contact@orthonov.paris
www.orthonov.com