Contents

1 cold pack

1 manual pump

1 freezer bag

1 lycra

Indications for use

Post-operative care, analgesic treatment, sports trauma (sprain, tear, etc...), after rehabilitation session...

Contraindications

Allergy to cold, insensibility to cold, Raynaud's syndrome, cryoglobulinemia, arteritis, diabetes, polyarthritis...

Precautions for use

The manufacturer makes no guarantee that the use of this product will prevent the occurrence of an injury. If user presents with characteristics that promote the risk of skin burns (e.g. malnourished patients, large smokers, fragile skin, insulin-dependent diabetics, etc.), close medical supervision is required and recommended.

This medical device should be used as prescribed by your surgeon or doctor or according to the recommendations for use of your duly authorized health professional (e.g. pharmacist, physiotherapist, orthotist).

Cold pack should never come into direct contact with the skin.

Do not puncture the cold pack.

In the event of a leak, do not pour the gel into a sewer-connected drainage system or into a stream.

For more information, contact Orthonov or visit www.orthonov.com

If the gel is in direct contact with the skin, wash immediately and thoroughly with water.

Contact your doctor if you have persistent irritation.

Keep out of the reach of children or the elderly unable to use this device.

Maintenance

The cold pack is washable by hand with a mild soap.

Do not immerse it in a bath of disinfectant detergent but spray it with a disinfectant detergent solution (ANIOS high surface disinfectant detergent is recommended).

Ensure the tap is in the F position.

This medical device is a regulated health product bearing the CE mark, under this regulation.

Before any use, please read this instruction carefully and respect the recommendations for use.

WARNINGS

In case of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your doctor, surgeon, or authorized health professional (e.g. pharmacist, physiotherapist, orthotist) as soon as possible, and contact Orthonov: +33 1 60 13 65 73 or contact@orthonov.com

Fabricant

WARNING
Manufacturer Caution,
Consult Accompanying
Documents

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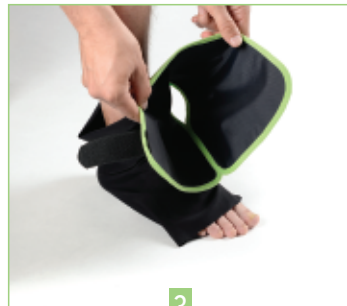
Put on the lycra clothes. The cold pack should not never be in direct contact with the skin.

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After 8 hours of freezing (between -18 and -22 °C), take out the cold pack.

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Distribute the gel if necessary. The green border must be under the sole of the foot.

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Place the foot in the cold pack. The heel must be in the hole.

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Fold down the left part of the cold pack on the foot, and take the velcro of the right side.

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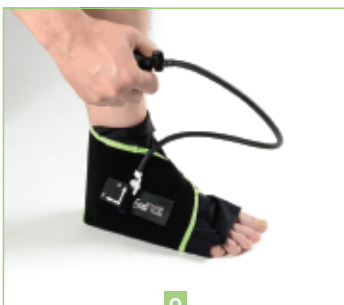
Fold down the right part of the cold pack on the left side and fix the velcro.

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Inflate (do not exceed 15 pressures). Turn the tap to F and unscrew the pump.

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After 20min, turn the tap to the O position and undo the strap. Wipe and replace the cold packs in the bag and store in the freezer..

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