Indications for use

Postoperative care of knee surgery, sports traumatology (sprain, tear, etc.), after rehabilitation session.
Analgesic, anti-oedematous, anti-haemorrhagic effects..

Contraindications

Allergy to cold, insensitivity to cold, Raynaud's syndrome or other vasospastic diseases, arteritis, cryoglobulinemia, sickle cell disease, arteriosclerosis, diabetes, polyarthritis, disorders of skin sensitivity or hypercoagulation, peripheral vascular diseases causing ischemia or weak local blood circulation.

Risks of use and precautions for use

This medical device must be used according to the medical prescription of your doctor or according to the recommendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.).

The manufacturer makes no warranty that use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements of the kit before use. Do not use if damaged.

Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or even people with fragile skin): close medical supervision is recommended.

Cold packs should never be in direct contact with the scar or skin. Systematically use the hypoallergenic lycra supplied.

Cold packs should be soft and frost free before use.

Do not heat or warm cold packs.

Respect the number of pressures indicated to inflate the pack. If tingling occurs in the limb, remove and deflate the pack immediately.

Do not pierce the cold packs, nor the manual pump and its connection pipe.

In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse.

Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.

In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water.

Contact your doctor if irritation persists.

Keep out of the reach of children and people unfit to use this device.

Cleaning

Cold packs and lycra are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath.

If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces).

Always place the tap in position F. Leave to air dry and lay flat.

For more information, contact Orthonov or visit the website www.orthonov.com

Warnings

This medical device is a regulated health product which follows under this regulation, the CE marking.

Before use, please read this manual carefully and follow the recommendations for use.

In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), and contact Orthonov: 01 60 13 65 73 or www.orthonov.com

Orthonov: 01 60 13 65 73 or www.orthonov.com

In the event of a serious reaction, contact the manufacturer and the competent authority in France.

Medical Device

CE Mark

Product traceability

Single patient multiple use

Batch Code

Optimal duration of use

Temperature limit

Consult instructions for use

WARNING
Manufacturer Caution,
Consult Accompanying
Documents

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Insert the cold packs in the airtight, dry freezer bag, before putting them in the freezer.

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After 8 hours of freezing (between -18 and -22 °C), take out the cold packs and distribute the gel if necessary.

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Place the posterior cold pack on a flat surface. The longest straps will be used for the thigh.

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Put on the lycra cloth and place the leg in order to center the back of the knee on the posterior cold pack from a above point of view.

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Place the anterior cold pack on the front of the knee. The green border must be on the thigh.

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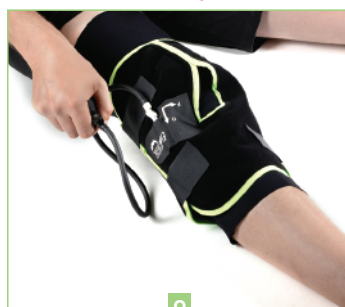
Fold down the posterior cold pack so that it overlaps the anterior cold pack and fix the velcro. If necessary, use the extensions.

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From now on, he leg must remain straight, if possible ankle above the knee level.

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Turn the tap to O and screw on the pump. Inflate (do not exceed 15 pressures). Turn the tap to F and unscrew the pump.

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After 20 min, turn the tap to the O position to release the remaining air. Undo the straps. Wipe and replace the cold packs in the bag and store in the freezer.

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